

New Copan eSwab for Bacteriology samples.



Pink cap Liquid Amies Single Swabs.

Use for Microbiology Culture and Sensitivity (MMCS) Investigations
Collection from: wound, skin, nose, throat, eyes, mouth, vaginal

Orange cap Liquid Amies Single Swabs.

Use for Ear swab culture and urethral swabs only.

IMPORTANT:

DO NOT USE FOR MRSA SCREENING OR VIROLOGY TESTING

MRSA SCREENING PACKS CAN BE ORDERED FROM PATHOLOGY SUPPLIES IF NEEDED



Remove swab from pack and take sample



Unscrew cap and insert swab into the liquid in the tube.



Break swab shaft at coloured breakpoint and leave swab in tube



Screw the cap tight onto the tube



Label tube with full patient ID or OrderComms label placed vertically on tube.

REMEMBER:

- **Liquid must remain in the tube - this is what is tested**
- **White swab must be in the tube - this is how we know a sample has been taken**
- **Always ensure samples are labelled fully.**